


How to use

Underlay Transtibial (BK)




User Guide

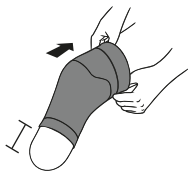

- 1 Slide the underlay over your residual limb, ensuring it sits **4-5 inches/10-13mm** from the distal end.

Adjust the height to suit your residual limb as needed.

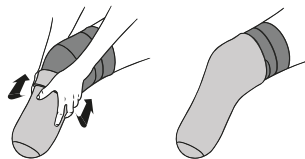

- 2 Roll your prosthetic liner as usual above the underlay ensuring the underlay extends above the liner.

Ensure that the breathing part is sitting above the liner extremity

OPEN

ETHNOCARE™

How it works

Underlay



Moisture wicking

The special knit fabric absorbs humidity from the skin and channels it upward toward the breathable mesh, where it can evaporate more easily.



Stay fresh treatment

Prevents the growth of odor-causing bacteria, keeping your product fresh even in sweat and humidity.



STAYFRESH™
POLYGENE FRESHNESS



STAYCOOL
BY POLYGENE



Breathable mesh



The breathable mesh promotes airflow and helps moisture evaporate quickly, keeping you dry and comfortable throughout the day.

Stay cool treatment



Proven to lower fabric temperature by 2-3°C for a refreshing feel.

Washing instructions



Do not use bleach to clean
Wash below 40°C / 104°F
Dry at low temperature
Do not iron.
Do not dry clean.