
ETHNOCARE

Redefine people's mobility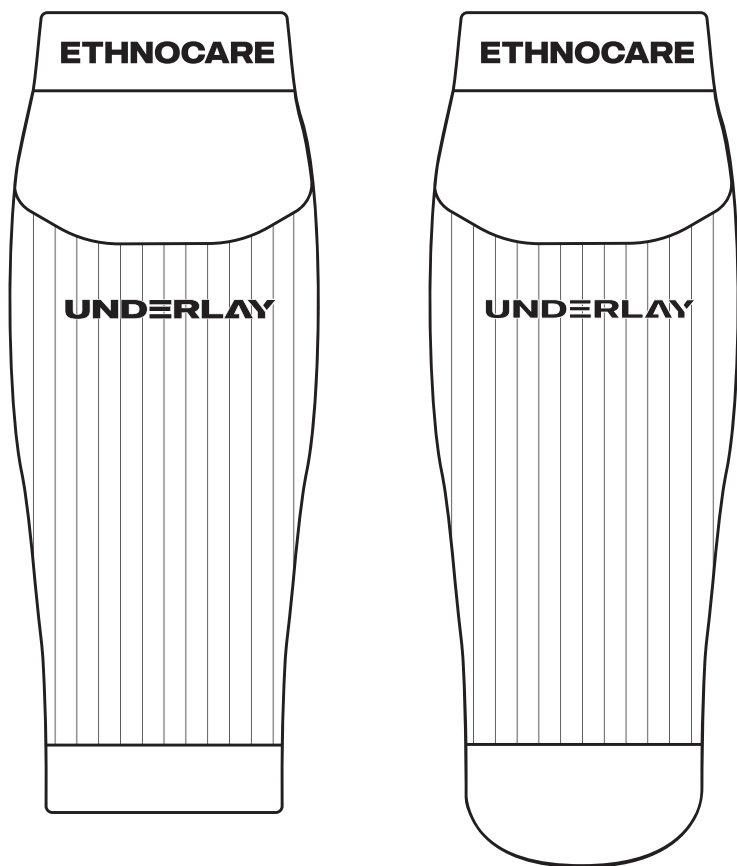

Instructions for use

UNDERLAY

TABLE OF CONTENTS

English — **3**

Français — **6**

Español — **9**

Deutsch — **12**

Norsk — **15**

Fig. 1

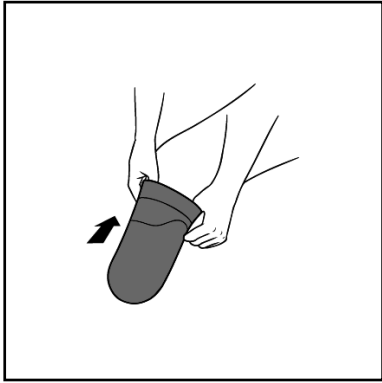
Fig. 2

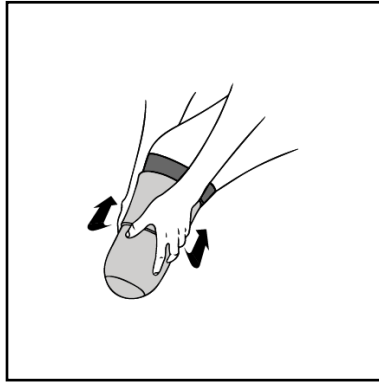
Fig. 3

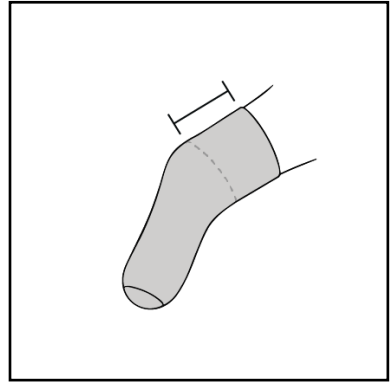
Fig. 4

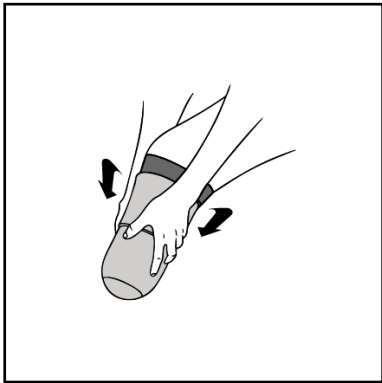
Fig. 5

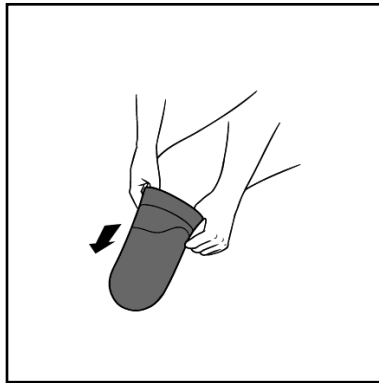
Fig. 6

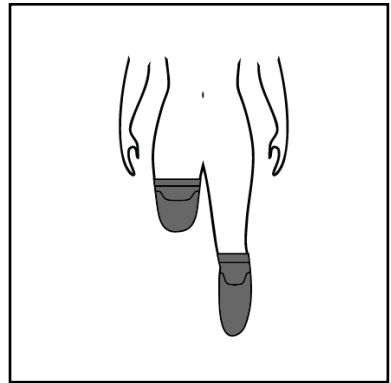
Fig. 7

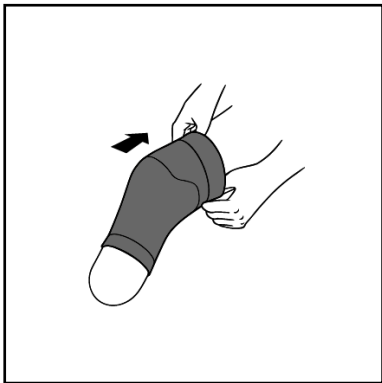
Fig. 8

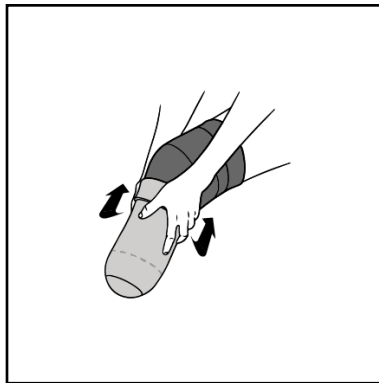
Fig. 9

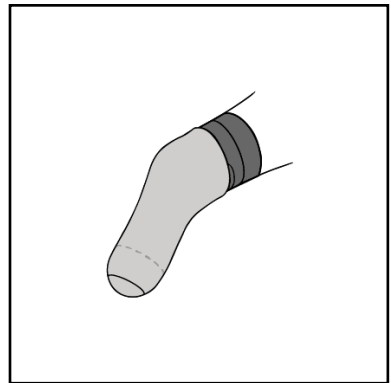
Fig. 10

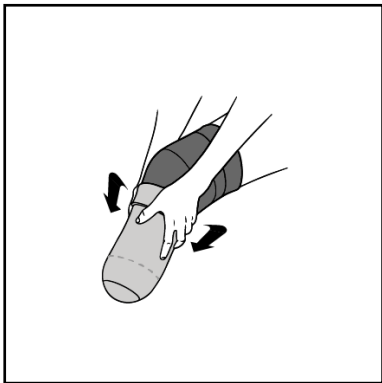
Fig. 11


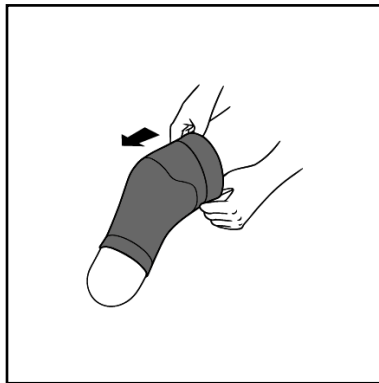
Fig. 12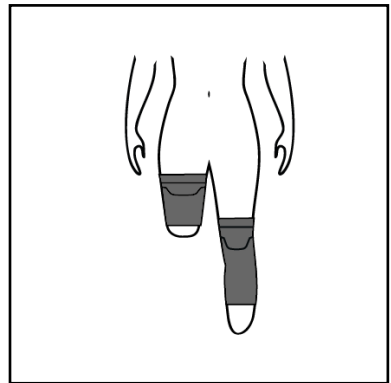

ENGLISH

READ THE FOLLOWING INSTRUCTIONS CAREFULLY IN THEIR ENTIRETY BEFORE USING THE DEVICE AND OBSERVE THE SAFETY INSTRUCTIONS. CORRECT INSTALLATION IS ESSENTIAL FOR THE PROPER FUNCTIONING OF THE DEVICE.

DEVICE DESCRIPTION & INTENDED USE

This device is a medical device for patients with lower limb amputations. It consists of a fabric sleeve with optional silicone designs to increase the grip. The device is treated with Polygiene, to reduce the growth of odor-causing bacteria. The device must be worn between the residual limb and the liner, to decrease the perceived humidity in the liner, and increase overall comfort for the user.

INFORMATIONS:

1. Please read this entire document carefully before using the device and follow the safety instructions.
2. A healthcare professional must instruct the patient or the caregiver how to use this device safely.
3. Contact the manufacturer, or your healthcare professional if you are a patient or a caregiver, if you have any questions about the device or if you experience any problems or complaints or incidents including any worsening medical conditions.
4. Please keep this document.

LIST OF MATERIALS

- Nylon
- Lycra
- Olefin
- Silicone
- Polygiene Fabric Treatment

USE OF SYMBOLS

Symbols	Symbols titles	Explanatory text
	Manufacturer	Indicates the medical device manufacturer
	Importer	Indicate the importer informations
	CE Mark	Indicate that the medical device is compatible with the European regulations.
	Packaging integrity	Indicates to not use the device if the packaging is damaged.
	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Catalog #	Indicates the manufacturer's catalog # so that the medical device can be identified.
	Quantity	Indicates the # of units per package.

	Consult instructions for use	Indicates the need for the user to consult the instructions for use.
	Medical device	Indicates that the item is a medical device.
	Single patient - Multiple use	Indicates that the medical device can be used multiple times, but only on one patient.
	Caution	Indicates that caution is necessary when operating the device to avoid undesirable outcomes.

- Symbols source : ISO 20417:2021

INSTRUCTION FOR USE:

Steps to Put the Device

NOTICE! Do not wear the device if you have a wounded or injured residual limb.

- Slide the Underlay over your residual limb, ensuring the:
 - Underlay sits 4-5 inches / 10-13 mm from the distal end of the limb for an open-ended device (fig. 7).
 - Underlay is not loose; the liner should fully cover the Underlay for a closed device (fig. 3).

NOTICE! Do not use your nails, as this could affect the integrity of the device.

- Don your prosthetic liner and socket as you normally would (fig. 2,3,8,9).

NOTICE! To minimize all risks of pressure points and feeling of numbness, stop wearing the device if it rubs on your residual limb or feels too tight, uncomfortable or painful.

Steps to Remove the Device

- Doff your prosthetic socket and liner as you normally would (fig. 4,11).
- Remove the Underlay as you would a standard sock (fig. 5,10).

Contraindications

- Do not use this device if you are allergic to the materials listed previously.
- Do not use this device on any type of amputation other than lower limb amputation.
- Do not use this device on a patient who is unable to communicate physical discomfort.
- Do not use this device if the patient has no prosthetic liner.

Useful Life

This device is expected to be used and cleaned daily, with a 45-day warranty.

GENERAL SAFETY INSTRUCTIONS

Warning

- Keep the device out of the reach of children. There is a choking hazard if the device is placed over the mouth and nose.

Caution

- Ensure that the device is properly placed and adjusted. Incorrect placement or fit can cause pressure and compression of blood vessels and nerves.
- Discontinue the use of the device if it doesn't fit properly, if the patient experiences pain, swelling, altered sensation or other abnormal reactions when using the device.
- The Underlay is not compatible with systems that do not use a prosthetic liner.
- The device is for single patient use and is not to be shared for use with other patients.
- Do not use the device if there are wounds or injuries on the residual limb.
- Follow the cleaning instructions.

Adverse effects

1. May cause skin redness.
2. May cause allergic reactions from the device materials.
3. May cause pressure points.
4. May cause soft tissue and skin tissue wounds (bruises).
5. May cause compression of blood vessels and nerves.

Notice

1. Do not modify the device or use it in any way other than intended.
2. Do not use this device if it is damaged and/or if the package has been opened.
3. If needed, take the necessary measures (e.g. replacement, check the manufacturer's service department, etc.).
4. Do not wear the device longer than a period of 16h per day.

CLEANING, MAINTENANCE AND STORAGE

1. Remove the device from the limb.
2. Clean the device after each use only with approved cleansing products including neutral soap or mild detergent that is pH balanced and free of fragrance, bleach and dye. Clean the device in warm water that is below 40°C (104°F).
NOTICE! The use of unsuitable detergents may cause the device to deteriorate and void the warranty.
NOTICE! Do not wring out the device to avoid damage.
3. Rinse the device thoroughly with clean warm water to remove all soap residues.
4. Insert a towel into the device and allow it to air dry. If you need to use a dryer, only use the low-temperature setting.
NOTICE! Do not iron or dry clean the device.
5. The device can be used only when it is completely dry.
NOTICE! Failure to follow these instructions will void the warranty.

WASTE DISPOSAL

The manufacturer accepts no liability for damages resulting from failure to comply with this document, in particular from improper uses, unintended uses, uses outside the devices specified use conditions or unauthorized modification of the device. The manufacturer's distribution company responsible in your country will provide you with further information on the conditions of the commercial guarantee.

LEGAL INFORMATION

The manufacturer grants a commercial warranty for this device from the date of fitting. The warranty period is 45 days. The commercial warranty covers proven defects in material, workmanship or construction. These defects must be reported to the manufacturer within the period of validity of the commercial warranty. Further information on the warranty terms and conditions can be obtained from the manufacturing company.

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Ethnocare Inc
Emergo Europe B.V.