

To determine the most suitable size for the patient: measure the circumference at 4 cm from the distal end over the skin. Then measure the length of the trimmed liner.

STEP 1

Choose suspension type.

Suspension


Compatible with all of the below

Pin

Lanyard

Sealing ring

Sleeve

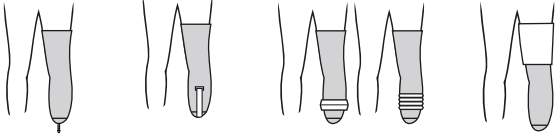


STEP 2

Measure circumference.

Circumference

Measure at 4 cm from the distal end (over the skin)

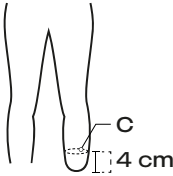


Measurement	23-28 cm	28-35 cm	35-42 cm
SKU	23	28	35

STEP 3

Measure length.

Length


Measure the length of the trimmed liner

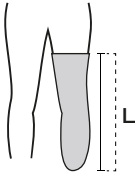

Measurement	27-37 cm	37+ cm
SKU	SH	LG

Suspension	Circumference	Length	SKU
Pin Lanyard	23-28 cm	27-37 cm	UDTT-23-SH
	28-35 cm		UDTT-28-SH
	35-42 cm		UDTT-35-SH
Sealing ring Sleeve	23-28 cm	37+ cm	UDTT-23-LG
	28-35 cm		UDTT-28-LG
	35-42 cm		UDTT-35-LG

Important:

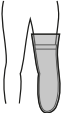

Underlay Closed must sit at least **5 cm (2 in)** below the top of the liner for suspension.




Compatible with pin, lanyard and sealing ring suspension systems.

To determine the most suitable size for the patient, measure two circumferences over the skin: at 10 cm from the distal end and just above the trimmed liner. Then measure the length of the trimmed liner.


STEP 1

Choose suspension type.

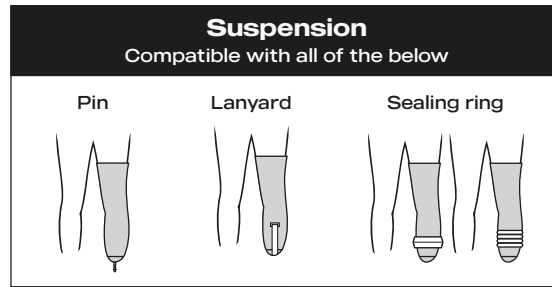

STEP 2


Measure distal circumference (C₁).

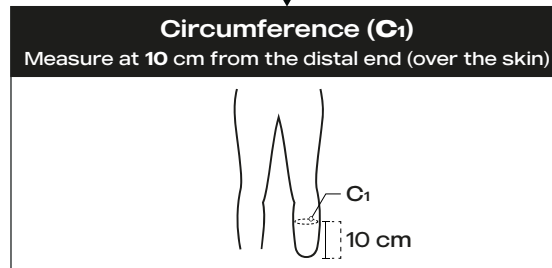

Measurement	23-28 cm	28-35 cm	35-42 cm
SKU	23	28	35

STEP 3

Measure proximal circumference (C₂).

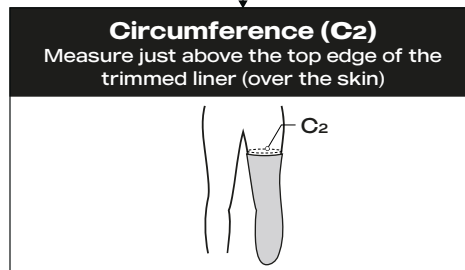


Starting with your C₁ measurement, measure C₂ to confirm whether the limb shape is compatible with Underlay Open.

If C₂ is much larger than C₁, then the limb shape is too conical - choose the Underlay Closed instead.

STEP 2	If C ₁ measured Size 23 , and:		If C ₁ measured Size 28 , and:		If C ₁ measured Size 35 , and:	
	C ₂ is less than 35 cm		C ₂ is less than 45 cm		C ₂ is less than 55 cm	
	C ₂ is greater than 35 cm		C ₂ is greater than 45 cm		C ₂ is greater than 55 cm	
Measurement	Keep Size 23		Keep Size 28		Keep Size 35	
SKU	Underlay Closed		Underlay Closed		Underlay Closed	

STEP 4

Measure length.

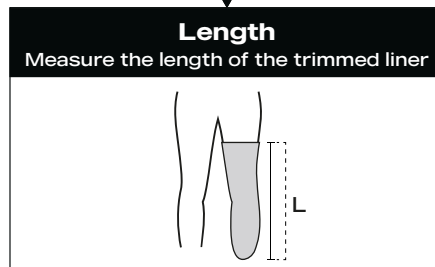

Measurement	23-28 cm	28-33 cm	33-38 cm	38+ cm
SKU	SH	MD	LG	XL

Suspension	Circumference (C ₁)	Length	SKU
Pin	23-28 cm	23-28 cm	UDTT-23-SH
	28-35 cm		UDTT-28-SH
	35-42 cm		UDTT-35-SH
Lanyard	23-28 cm	28-33 cm	UDTT-23-MD
	28-35 cm		UDTT-28-MD
	35-42 cm		UDTT-35-MD

Suspension	Circumference (C ₁)	Length	SKU
Pin	23-28 cm	33-38 cm	UDTT-23-LG
	28-35 cm		UDTT-28-LG
	35-42 cm		UDTT-35-LG
Lanyard	23-28 cm	38+ cm	UDTT-23-XL
	28-35 cm		UDTT-28-XL
	35-42 cm		UDTT-35-XL

Important:

